

SMS PIPELINE HOSE - INCORRECT ASSEMBLY

<u>Part Nos.</u>	SMS0062386	O2 Hose Assembly BS5682/NIST 3m
	SMS0062387	Air-4 Hose Assembly BS5682/NIST 3m
	SMS0062388	O2 Hose Assembly BS5682/NIST 30"

Your Order No.

Lot number: EDD

BACKGROUND

We have had a report of an incorrectly assembled MEDICAL AIR (MA4) pipeline hose. We believe that the problem is confined to the listed part numbers with the lot number EDD. The pipeline hoses would have been supplied through Fisher & Paykel Healthcare.

Customer supply records indicate that you were supplied hoses under the above order number. We are asking for your cooperation to confirm that there are no more incorrectly assembled hoses in circulation.

ACTION

Please locate any of the listed hoses with the prefix EDD in your establishment and confirm that the end fittings are correct.

Please complete the attached faxback even if you find no incorrectly assembled pipeline hoses.

For further advice contact Raymond Day on 01279 406000



Check for correct gas NIST fitting here.

AIR-4 hose shown should have Air-4 inscribed where indicated.

An oxygen hose (white) should be fitted with an oxygen NIST fitting inscribed O2.

Location of Lot Code.

Medical Device Correction Response Form

Please complete below and return this form to us : Attention Raymond Day SMS Technologies

FAX 01279 406001

or by post to address below

Raymond Day
SMS Technologies
St James Centre
East Road
HARLOW
Essex
CM20 2SX

Note: A response is requested even if you do not have any incorrectly assembled pipeline hoses.

Hospital: _____

Name _____

Phone: _____

E-mail address: _____

Signed: _____

Position: _____

Date: _____

Please tick appropriate box below and fax back to SMS Technologies

Correct Hose Connections on all hoses with lot EDD

Incorrect Hose(s) Connection Found